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Destinazione d'uso

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en INSTRUCTIONS FOR USE

Intended use

The CPAP breathing bag is intended for use during CPAP ventilation when wishing to maintain the pressure value constant as the volume inspired by the patient changes.

Restrictions of use

- The device must be used by qualified and trained medical/nursing staff aware of the possible side effects of positive pressure ventilation.
- To be used for administration of air and oxygen.
- If used properly, the device can be used continuously for a maximum of 7 days after which it must be replaced.

Instructions for use

Remove the device from the package. Attach the connector to the ventilation circuit. Check that the connection is secure before proceeding with administration of the therapy gases. If condensate forms, empty the device by opening the drain cap.

Warnings and precautions

⚠ Single use device. Reuse is not allowed since it could cause cross contamination in patients. Moreover, materials which make part of the device could be damaged after cleaning/disinfection and therefore they could not guarantee required performances, therefore creating a safety risk for patients.

- In the event of an evident change in performance or inadequate performance of the device, it is recommended to replace it.
- Monitor the clinical parameters of the patient during the treatment.
- It is unadvisable to use humidified ventilation gas since this would cause the formation of condensation on the walls of the device.

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- The device can be used for maximum 7 days Depending on the clinical picture of the patient the medical/nursing staffs are responsible for defining the need for more frequent replacement of the device.

- Non-sterile. Do not sterilize.
- Do not use on another patient.
- Expiry: 5 years provided that the packaging is undamaged and if stored in normal storage conditions [-20°C/+50°C).
- Dispose of the materials immediately after use conforming to the current laws and regulations.
- Use only with devices that have connections in compliance with the ISO 5356-1 standards.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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